

## Prontosan<sup>®</sup> Debridement Pad

SOFT MECHANICAL REMOVAL OF SLOUGH AND DEBRIS

# Experts in Wound Cleansing

## A NEW ADDITION TO COMPLETE THE PRONTOSAN® RANGE

Designed for soft mechanical debridement of slough and debris. The Prontosan® Debridement Pad removes and binds slough and debris and assists with effective wound bed preparation - new granulation tissue is left intact.

### FEATURES

- Good cleansing and debridement due to microfiber technology
- Soft debridement, no tissue irritation
- Unique droplet shape to allow debridement of cavities and areas difficult to reach
- Blister packaging to allow safe and aseptic soaking of the pad prior to use
- Produces good results even with scaly and necrotic coatings, if they are subject to prior autolytic treatment

### INNOVATIVE SHAPE - DESIGNED FOR SUCCESS




### Micro Technology, Maximum Cleansing

The Prontosan® Debridement Pad uses the latest technology in cleansing - microfibres. Microfibres are very small and utilise 'electrostatic forces' to attract even the smallest particles to them.

Each microfibre has a multiple strand structure, allowing for many more particles of slough and debris to be **removed** from the wound bed and **bound** to each microfibre within the pad; not just brushed away as with traditional, much larger, monofilament fibres.

There are **millions of microfibres** in each Prontosan® Debridement Pad for magnified debridement power.

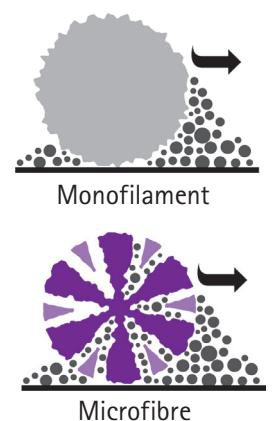

BEFORE AND AFTER USING THE PRONTOSAN® DEBRIDEMENT PAD



Before debridement



After a single debridement



3 weeks post debridement

HOW TO USE FOR BEST RESULTS

- 1 Open the packaging, using the integrated tray to moisten (with Prontosan Solution\*), covering the microfibre side of the pad
  - 2 Applying light pressure, use circular or sweeping motions over areas of slough and debris
  - 3 Irrigate (with Prontosan Solution\*) to cleanse
  - 4 Apply Prontosan Gel X to help prevent biofilm formation and apply dressings as appropriate\*\*
- TIP**
- B. Braun branded side faces away from the wound bed
  - No need to remove emollient prior to use
  - Prontosan® Solution (as soak or rinse) can be used beforehand



Before use of Prontosan® Debridement Pad



After use of Prontosan® Debridement Pad



Part of the Prontosan® Range  
Please visit [www.bbraun.com/prontosan](http://www.bbraun.com/prontosan)  
for more information



\*Saline can be used

\*\*Depending on local policy

# Prontosan® Debridement Pad

## ORDERING INFORMATION

Product Size	Art-No
Pack of 3 Pads	3908456
Pack of 10 Pads	3908457



B. Braun Medical AG | Infection Control | Seesatz 17 | CH-6204 Sempach

Phone +41 58 258 50 00 | Fax +41 58 258 60 00 | [info.bbmch@bbraun.com](mailto:info.bbmch@bbraun.com) | [bbraun.com/infection-prevention](http://bbraun.com/infection-prevention)